




Pairing your iOS smart devices with your blood pressure (BP) device

Once you've downloaded the latest version of the OMRON connect app from the App store, you'll need to unpair your BP monitor from the Bluetooth® Settings on your iOS smart devices.


1

- Open **Settings**  on your smart device.
- Open **Bluetooth®** by tapping on it.
- Search **MY DEVICES** for a device that starts with **HEM** or **BP**.
- Tap on the  icon to the right of the device.
- Tap on **Forget This Device**.
- Confirm by tapping on the **Forget Device** button.


c

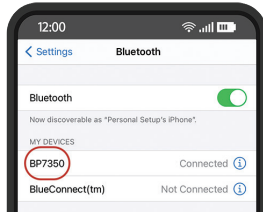

e

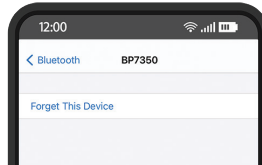


f

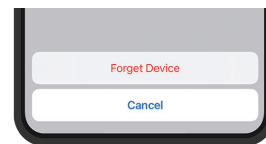


The *Bluetooth®* word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc. and any use of such marks by OMRON Healthcare, Inc. is under license.

2

Now open the OMRON connect US/CAN App.

- Tap on the  icon, then the **Profile** icon.
- Now select **Connected Devices** and tap the **Add new device** button.
- Select **All OMRON Blood Pressure Devices**.
- Tap on the **Next: Pairing** button.

b

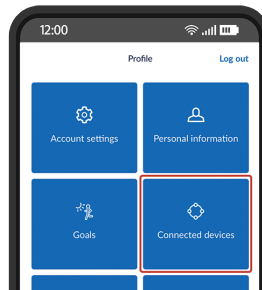

c




d

If you have questions or issues, please visit our **Frequently Asked Questions** at <https://omronwellness.com/mobilefaq/>, contact customer service at **1 (800) 634-4350** or email us at omronhealthcaresupport@omron.com.

Pairing your iOS smart devices with your blood pressure (BP) device (cont'd)

3


To place your monitor in pairing mode, **press and hold the Transfer** button on your blood pressure monitor for three to five seconds. A **capital letter P** will display on your monitor.

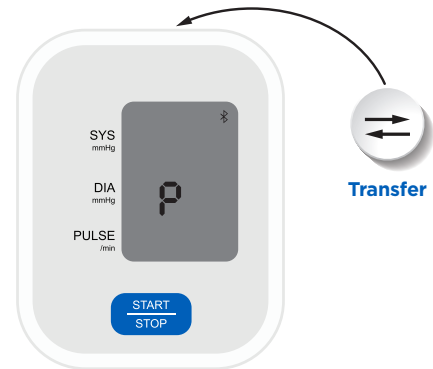

4

- Return to your smart device and press the **Next: Begin pairing** button.
- Click on the photo of your device.**
- Confirm your Bluetooth Pairing Request by clicking on **Pair**.

b

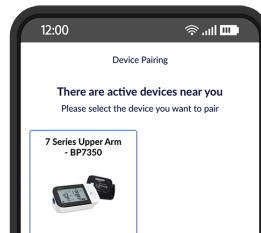

5

- If your device allows two users, you will be asked to select the User number. If you, alone, will be using the monitor, select 1. If User 1 is already taken by another household member, select User 2.
- Once selected, tap the **Confirm** button. If both user 1 and 2 are already assigned on the device, you will be asked to confirm your selection. Tap on the "Yes" button to confirm.
- Congratulations!** You are now connected and can start using the app with your BP device.

b



c

