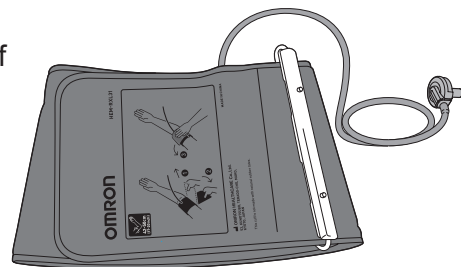
# Extra-Large Cuff

## Blood pressure monitor cuff

### HEM-RXL31

HEM-RXL31  
9301851-0C

Issue Date: 2022-01-11

Thank you for purchasing the OMRON cuff HEM-RXL31. Please refer to the instruction manual of your Blood Pressure Monitor.

**Description:**

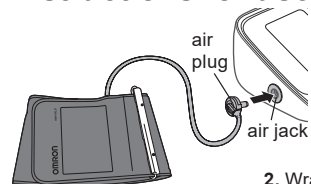
This Omron blood pressure cuff consists of an inflatable bladder with sleeve (cuff) that is used with a blood pressure monitor to determine a patient's blood pressure. This cuff should only be used with Omron blood pressure monitors that have "HEM-RXL31" indicated on the side of the air jack. Arm circumference for this cuff is 42 to 50 cm (17" to 20").

Please follow this instruction manual thoroughly for your safety before using the device. Please keep for future reference. For specific information about your own blood pressure, CONSULT WITH YOUR PHYSICIAN.

### Instructions for use

Remove tight fitting clothing or a tightly rolled up sleeve from your left upper arm. Do not place the arm cuff over thick clothing.

1. Connect the air plug to the unit.

Note: Only use this cuff with an OMRON Blood Pressure Monitor that has "HEM-RXL31" indicated on the side of the air jack.

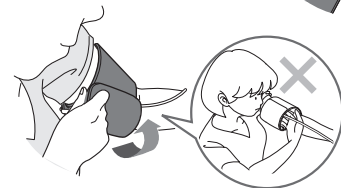
2. Wrap the arm cuff firmly in place around your left upper arm.

Place the edge of the cuff at the crease of the elbow.

3. Securely fasten the cuff with the fabric fastener.

Notes: 1. Remain still and do not talk during the measurement.  
2. Incorrect use may cause an inaccurate measurement.

#### Taking measurements on the right arm

The air tube will be at the side of your elbow. Be careful not to rest your arm on the air tube.

Note: Blood pressure can differ between the right arm and left arm, and may result in differences in readings. OMRON recommends to always use the same arm for measurement. If the readings between both arms differ greatly, please check with your physician as to which arm to use for your measurement.

**Caution:** Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.

- Use the same arm cuff as written on the side of the product by the air jack. Do not use any other type of cuff.
- Do not wash the cuff.
- Do not wrap the cuff on an injured arm or an arm under medical treatment.
- Do not use the cuff if it is damaged.
- Make sure the connections are tight. An air leak will lead to incorrect measurements.
- Ensure that the cuff tube does not become wrapped around the patient's arm or other body part.
- Do not connect this cuff to unspecified products. This may cause an accident.
- If a physician has indicated that the patient has hemorrhagic diathesis or hypercoagulability, check the condition of the arm after measurement and if any abnormalities are noted, contact your physician.
- Remain still and do not talk during the measurement.
- If used with even a slight amount of liquid in the cuff, the liquid may enter the monitor to which the cuff is connected and cause the monitor to malfunction.

**Warning:** Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

- Contains small parts that may cause a choking hazard if swallowed by infants, toddlers, or children.

### Specifications

Product Description	Blood pressure monitor cuff
Model	HEM-RXL31
Arm Circumference	42 to 50cm (17" to 20")
Dimensions	Approximately 153mm x 742mm (air tube : 750mm) (6" x 29 1/4" (air tube : 29 1/2" )
Operating Conditions	10~40°C(50~104°F),15%~90%RH(non-condensing),700~1060hPa
Storage Conditions	-20~+60°C(-4~+140°F),10%~95%RH(non-condensing),700~1060hPa
Durable Period (Service Life)	1 year

### Maintenance

- Use a soft and dry cloth, or a soft and moistened cloth and neutral soap to clean the arm cuff.
- Do not use gasoline, thinners or similar solvents to clean the arm cuff.
- Do not store the arm cuff in locations exposed to extreme temperatures, humidity, direct sunlight, dust or corrosive vapors such as bleach.
- Do not bend or crease the air tube excessively.
- Dispose of the cuff and components according to applicable local regulations. Unlawful disposal may cause environmental pollution.

### Limited Warranty

This warranty only applies to users in the U.S.A and Canada. Your HEM-RXL31 cuff is warranted to be free from defects in materials and workmanship appearing within 1 year from the date of purchase, when used in accordance with the instructions provided with this cuff. The above warranty extends only to the Telehealth service provider. We will, at our option, replace without charge any arm cuff covered by the above warranty. Replacement is our only responsibility and your only remedy under the above warranty. **THE FOREGOING IS THE SOLE WARRANTY PROVIDED BY OMRON IN CONNECTION WITH THIS PRODUCT, AND OMRON HEREBY DISCLAIMS ANY OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. IMPLIED WARRANTIES AND OTHER TERMS THAT MAY BE IMPOSED BY LAW, IF ANY, ARE LIMITED IN DURATION TO THE PERIOD OF THE ABOVE EXPRESS WARRANTY. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE LIMITATION MAY NOT APPLY TO YOU.**

**OMRON SHALL NOT BE LIABLE FOR LOSS OF USE OR ANY OTHER SPECIAL, INCIDENTAL, CONSEQUENTIAL OR INDIRECT COSTS, EXPENSES OR DAMAGES. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.**

This warranty provides you with specific legal rights, and you may have other rights that vary by jurisdiction. Because of special local requirements, some of the above limitations and exclusions may not apply to you. Should warranty service be required, please contact your Telehealth service provider.

Symbols description			
	CE Marking		Batch code
	Medical device		Temperature limit
	Humidity limitation		Atmospheric pressure limitation
	Identifies the type of cuff that is compatible with the blood pressure monitor		Catalogue number
	Date of Manufacture		Consult instructions for use
	Manufacturer		Authorized representative in the European Community

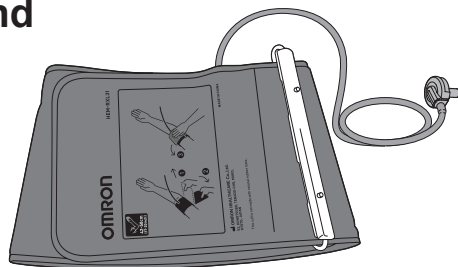
This cuff fulfills the provisions of EN1060 Non-invasive sphygmomanometers Part 1: General Requirements and Part 3: Supplementary requirements for electromechanical blood pressure measuring systems. This OMRON product is produced under the strict quality system of OMRON HEALTHCARE Co., Ltd., JAPAN. Please report to the manufacturer and the competent authority of the Member State in which you are established about any serious incident that has occurred in relation to this device.

	<b>Manufacturer</b> OMRON HEALTHCARE Co., Ltd. 53, Kunotsubo, Terado-cho, Muko, KYOTO, 617-0002 JAPAN
	<b>Distributor</b> OMRON HEALTHCARE, INC. 2895 Greenspoint Pkwy. Hoffman Estates, IL 60169 USA OmronHealthcare.com Toll free US & CAN 1-800-634-4350
	<b>Asia Pacific HQ</b> OMRON HEALTHCARE SINGAPORE PTE LTD. www.omronhealthcare-ap.com
	<b>EU-representative</b> OMRON HEALTHCARE EUROPE B.V. Scorpius 33, 2132 LR Hoofddorp, THE NETHERLANDS www.omron-healthcare.com
	<b>Importer in EU</b>

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