## IMPORTANT SAFETY INSTRUCTIONS SAVE THESE INSTRUCTIONS

### AC Adapter Model HEM-ADPTW5

Thank you for purchasing the Omron® HEM-ADPTW5 AC Adapter.

 Please read this instruction manual thoroughly before using the unit.

Please keep for future reference.


#### -DANGER-

#### TO REDUCE THE RISK OF FIRE OR ELECTRIC SHOCK, CAREFULLY FOLLOW THESE INSTRUCTIONS.

-  For connection to a power supply not in the U.S.A. or Canada, use an attachment plug adapter of the proper configuration for the power outlet.
-  The power unit (AC Adapter) is intended to be correctly oriented in a vertical or floor mount position.

#### TO CONNECT THE AC ADAPTER

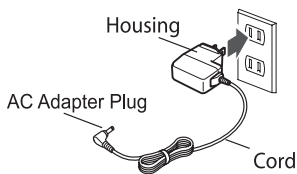- Insert the AC Adapter Plug into the AC Adapter Jack on the Omron device.
- Plug the AC Adapter into a 120V AC outlet (60 cycles).

#### TO DISCONNECT THE AC ADAPTER

- Hold and pull the Housing to remove the AC Adapter from the electrical outlet. Do not remove by pulling the cord.
- Remove the AC Adapter plug from the Omron device.

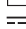
#### WARNING

-  Do not plug or unplug the power cord into the electrical outlet with wet hands.
-  Do not overload power outlets. Plug the device into the appropriate voltage outlet.

#### NOTES

Operating temperature / humidity / air pressure :  
50°F to 104°F (10°C to 40°C) / 15 to 95% RH / 700 to 1060hPa

Storage temperature / humidity :  
-4°F to 140°F (-20°C to 60°C) / 10 to 95%RH

 : Class II equipment IP21 : Ingress Protection rating  
 : Direct current  : Alternating current

Manufactured for :  
OMRON HEALTHCARE Co., Ltd.  
53 Kunotsubo, Terado-cho,  
Muko, Kyoto, 617-0002 JAPAN

Distributed by :  
OMRON HEALTHCARE, INC.  
2895 Greenspoint Pkwy.  
Hoffman Estates, IL 60169 USA  
Made in Philippines

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