

Wireless Pad Instruction Manual

Model: PMWPAD-L / PMWPAD-M

(for use with Avail™ Wireless Dual Channel TENS only)

OmronHealthcare.com


Single Patient Use Only

3125476-5C

INTENDED USE

The Wireless Pad is intended for use as a disposable, conductive adhesive interface between the patient's skin and the electrical stimulator. This pad should ONLY be used with Avail™ Wireless Dual Channel TENS (PM601), and applied on normal, healthy, dry, clean skin of adults.

SAFETY INFORMATION

 Please read this instruction manual thoroughly before using the pad.

For more details, please read the device instruction manual.

CONTRAINDICATION

Do not use this pad if you have a cardiac pacemaker, implanted defibrillator or other implanted metallic or electronic device. Such use could cause electric shock, burns, electrical interference or death.

▲ WARNINGS Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

▲ CAUTIONS Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.

WARNINGS

▲ Do not place the pads across your chest or near your heart because the introduction of electrical current into the chest may cause rhythm disturbances to your heart.

▲ Do not place the pads near or on skin with poor circulation, cancerous lesions, sensitive or diseased skin, open wounds, rashes, swollen, red, infected or inflamed areas or skin eruptions such as phlebitis, thrombophlebitis and varicose veins.

▲ Do not place the pads over your neck because this could cause severe muscle spasms resulting in closure of your airway, difficulty in breathing or adverse effects on heart rhythm or blood pressure.

▲ Do not apply the pads when bathing, showering, sleeping, during exercise, while sweating or in high humidity.

▲ KEEP THIS PAD OUT OF THE REACH OF INFANTS, TODDLERS, AND CHILDREN.

CAUTIONS

▲ Do not use this device to treat one region for extended periods of time. The long-term effects of electrical stimulation are unknown.

▲ Use caution if stimulation is applied over areas of skin that lack normal sensation.

▲ Do not apply the pads with wet hands and do not apply the pads that are wet.

Adverse Reactions

- You may experience skin irritation and burns beneath the stimulation electrodes applied to your skin.
- You may experience headache and other painful sensations during or following the application of electrical stimulation near your eyes and to your head and face.

HOW TO USE THE PAD

1. Attach the device onto the pad.
2. Remove and discard the plastic film from the back of the pad.
3. Place the pad only on your skin.

Notes:

- The pads should be applied only to normal, intact, clean, healthy skin that is not experiencing any swelling or inflammation.
- Pad is for Single Patient Use Only. Never share the pad with another person.
- Do not use the pads after expiration date.
- With proper storage and use, you may extend the life of your pad to 30 uses. The life of the pad may vary due to skin condition and storage condition.

WHEN TO REPLACE THE PAD

Replace the pad when it no longer sticks to your skin or if more than 25% of the pad surface does not make contact with your skin.

CLEANING THE PAD

1. Remove the device from the pad before cleaning the pad.
2. If you want to clean the ADHESIVE side of the pad, wipe the surface with a soft cloth moistened with water. If you want to clean the NON-ADHESIVE side of the pad, wipe the surface with a soft cloth moistened with water or a neutral detergent, then wipe the surface with a dry soft cloth to remove extra water.
 - Do not wash the pad with running water.

STORAGE

- Never bend or fold the pad.
- Storage temperature: 32 to 104°F (0 to 40°C)
- Operating temperature: 50 to 104°F (10 to 40°C)
- Relative humidity: 30 to 80%
- With proper storage, you may extend the life of your pad to 30 uses.

If you have any questions or need assistance, please call 1-800-634-4350 or visit OmronHealthcare.com.

Manufactured for:

OMRON HEALTHCARE Co., Ltd.

53, Kunotsubo, Terado-cho, Muko, Kyoto, 617-0002 JAPAN

Distributed by:

OMRON HEALTHCARE, INC.

1925 West Field Court, Lake Forest, IL 60045 U.S.A.

Made in Japan

© 2017 OMRON HEALTHCARE, INC.