

CLEAN PARTS BEFORE FIRST USE

⚠ WARNING

- Failure to properly clean the COMP A-I-R components may cause bacterial growth, serious illness from contamination, compromised medication delivery.
- Clean the parts of this device before using this device for the first time and also when the device has not been used for an extended period of time.
- Clean the parts of this device after each use.
- After cleaning, allow the parts to dry before reassembling. To avoid contamination, store the device and parts in a clean and safe location.
- DO NOT leave the cleaning solution on the parts. Rinse the parts with clean tap water after cleaning.


⚠ CAUTION

- Dry the parts by air drying only. DO NOT use any equipment to dry the device parts.
- To prevent damage to the device, DO NOT immerse the main unit in any liquid, DO NOT wash or rinse any of the parts under strong running water.
- To prevent damage to the device, DO NOT clean the main unit using abrasive cleaners or any type of chemical.
- NEVER clean with benzene or thinner.


Cleaning the Device Before First Use

1 Remove the inhalation top and vaporizer head from the medication tank.

Remove any accessories (mouthpiece, mask or child accessory) from the nebulizer kit if already attached.

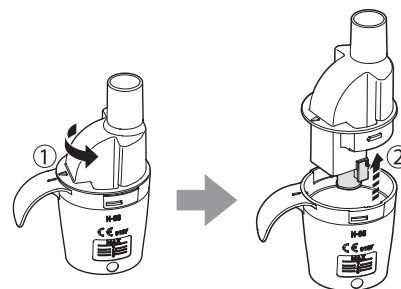

2 Remove the vaporizer head from the inhalation top.

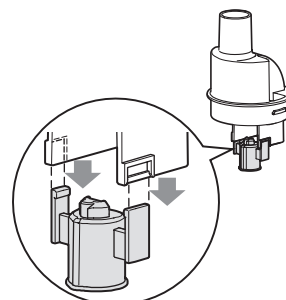

3 Wash the parts.

Wash the inhalation top, medication tank and accessories (mouthpiece, mask or child accessory) in warm water with mild (neutral) detergent and rinse all pieces thoroughly with clean hot tap water. Wash the vaporizer head with running tap water.


⚠ WARNING

- DO NOT leave the cleaning solution on the parts. Rinse the parts with clean tap water after cleaning.
- DO NOT immerse the main unit or the AC adapter in water or other liquid.

Wash in warm water with mild (neutral) detergent



Wash with running tap water

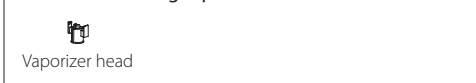

DO NOT wash




4 Dry the parts.

Allow to air dry in a clean environment.

⚠ WARNING

After cleaning, allow the parts to dry before reassembling. To avoid contamination, store the device and parts in a clean and safe location.

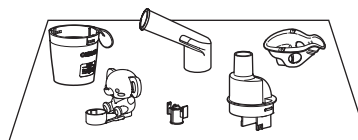

Cleaning the Device After Each Use

The following items should be disassembled and cleaned after each use in order to prevent any remaining medication in the tank from drying which will result in ineffective nebulization.

- Nebulizer kit (inhalation top, medication tank and vaporizer head)
- Mouthpiece
- Mask

Refer to steps above to clean each item.

Cleaning the Main Unit and Air Tube


Clean the casing of the main unit and the air tube by using a soft cloth moistened with water or a mild detergent. Do not use abrasive cleaners. Dry the casing and air tube immediately using a soft clean cloth.

⚠ WARNING

- DO NOT immerse the main unit or the AC adapter in water or other liquid.

⚠ CAUTION

- Unplug the AC adapter before cleaning this device.
- DO NOT insert any object into the main unit.

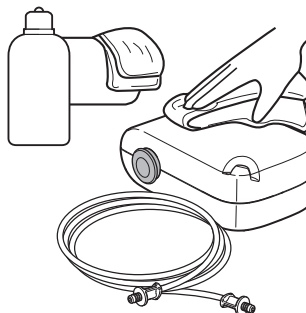

Cleaning the Device Weekly

The following items should be cleaned by boiling once a week. If the parts are heavily stained, replace them with new ones.

- Nebulizer kit (inhalation top, medication tank and vaporizer head)
- Mouthpiece

Parts may be boiled between 15 to 20 minutes.

After boiling, carefully remove the parts, shake off excess water and allow to air dry in a clean environment.

