Easy Wrap ComFit™ Cuff
Brazalete de colocación fácil ComFit™
Brassard facile à enrouler ComFit™
CFX-WR17

| | | |
|-------------------------------|---|----|
| • <i>Instructions for use</i> | 2 | EN |
| • <i>Instrucciones de uso</i> | 4 | ES |
| • <i>Mode d'emploi</i> | 6 | FR |

Thank you for purchasing the OMRON Easy Wrap ComFit™ Cuff (CFX-WR17). Arm circumference for this cuff is 9" to 17" (22 to 42 cm). Please refer to the original instruction manual of your Blood Pressure Monitor.

Please read this instruction manual thoroughly before using the unit.
Please keep for future reference.


Caution: Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.

- Use only the approved OMRON Blood Pressure Monitor with this cuff. Works with OMRON Advanced Accuracy blood pressure monitors compatible with ComFit™ Cuff.
- Use the same arm cuff model as written next to the air jack. Do not use any other type of arm cuff.
- Do not wash the cuff or make it wet.

Instructions for use

Remove tight-fitting clothing or tight rolled up sleeve from your left upper arm.


Do not place the arm cuff over thick clothes.

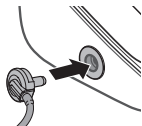1. Insert the air plug into the air jack securely.

2. Apply the arm cuff to your left upper arm.

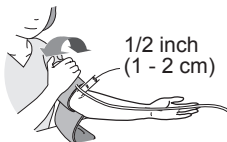The bottom edge of the arm cuff should be 1/2 inch (1 - 2 cm) above the elbow. Air tube is on the inside of your arm and aligned with your middle finger.

3. Secure closed with the fabric fastener.

Taking measurements on the right arm

The air tube will run along the inside of your arm. Be careful not to rest your arm on the air tube.

Note: The blood pressure can differ between the right arm and the left arm, and the measured blood pressure values can be different. OMRON recommends to always use the same arm for measurement. If the values between both arms differ substantially, please check with your physician which arm to use for your measurement.

- Use a soft and dry cloth, or a soft and moistened cloth and neutral soap to clean the arm cuff.
- Do not use gasoline, thinners or similar solvents to clean the arm cuff.
- Do not store the arm cuff in extreme temperatures, humidity, direct sunlight, dust or corrosive vapors.
- Do not bend or crease the air tube excessively.

Maintenance

Limited Warranty

Your CFX-WR17 cuff is warranted to be free from defects in materials and workmanship within 1 year from the date of purchase, when this cuff is used in accordance with the instructions provided with this cuff. The above warranty extends only to the original retail purchaser. We will, at our option, replace without charge any arm cuff covered by the above warranty. Replacement is our only responsibility and your only remedy under the above warranty. To obtain warranty service contact Customer Service by calling 1-800-634-4350 for the address of the inspection center, the return shipping and handling fee, and further instructions.

THE FOREGOING IS THE SOLE WARRANTY PROVIDED BY OMRON IN CONNECTION WITH THIS PRODUCT, AND OMRON HEREBY DISCLAIMS ANY OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. IMPLIED WARRANTIES AND OTHER TERMS THAT MAY BE IMPOSED BY LAW, IF ANY, ARE LIMITED IN DURATION TO THE PERIOD OF THE ABOVE EXPRESS WARRANTY.

OMRON SHALL NOT BE LIABLE FOR LOSS OF USE OR ANY OTHER SPECIAL, INCIDENTAL, CONSEQUENTIAL OR INDIRECT COSTS, EXPENSES OR DAMAGES.

This warranty provides you with specific legal rights, and you may have other rights that vary by jurisdiction. Because of special local requirements, some of the above limitations and exclusions may not apply to you.

| | |
|--|--|
| Product Description | Blood pressure monitor cuff |
| Model | CFX-WR17 |
| Arm Circumference | 9" - 17" (22 - 42 cm) |
| Dimensions | Approximately 5 3/4" × 21" (air tube: 29 1/2") (145 mm × 532 mm (air tube: 750 mm)) |
| Operating temperature / humidity | 50°F to 104°F (10°C to 40°C) / 15 to 90% RH |
| Storage temperature /humidity / air pressure | -4°F to 140°F (-20°C to 60°C) / 10 to 95% RH / 700 to 1060 hPa |

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© 2021 OMRON HEALTHCARE, INC.
EN Manufactured for: OMRON HEALTHCARE Co., Ltd.
ES Fabricado para: OMRON HEALTHCARE Co., Ltd.
FR Fabriqué pour : OMRON HEALTHCARE Co., Ltd.