

# Focus Knee Replacement Band Instruction Manual

Model: PMKBAND-M/PMKBAND-L

(for use with Focus TENS unit only)

**OMRON.**

HV-KBAND-MZ/HV-KBAND-LZ

2889237-8B

## Intended Use

The device is intended for the relief of pain associated with sore or aching muscles of the lower extremities (leg) due to strain from exercise or normal household work activities. It is also intended for the use of symptomatic relief and management of chronic, intractable pain associated with arthritis.

Environment of Use: Clinics, hospital and home environments

Patient Population: Adult

## Device Description

The Focus Knee Replacement Band is intended for use as a fixable interface between the patient's skin and the electrical stimulator with pad. This band should ONLY be used with the Focus (PM710-M/PM710-L).

## Important Safety Information

### WARNING

**Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.**

- DO NOT use this device if the circumference below your knee is out of the specified range of the band size.

### CAUTION

**Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient, or cause damage to the equipment or other property.**

- DO NOT use the device when the band is damaged or the hook and loop fasteners lose cohesion.
- DO NOT fasten the band too tight.

## How to Use the Band

1. Put the connector of the pad through the band port.
2. Attach the device onto the band.
3. Remove the protective sheet from the loop fastener.
4. Place the band on your bare skin directly.

## How to Clean the Band

1. Remove the device and pad from the band.
2. Hand-wash and air-dry it in a shaded area.

**Note:**

- Wash the band by hand according to the instructions indicated on the band.
- Do not soak the band in water for long periods of time.
- Align the shape of the band while letting it dry.
- Repeated washing may gradually change the color and sheen of the band.
- Do not use any sterilization chemicals, disinfectants or softeners.
- Replacing the band every 6 months is recommended.
- To purchase a new pad or band, call 1-800-634-4350 or visit [OmronHealthcare.com](http://OmronHealthcare.com).

## Storage

To extend the life of the band, use and store properly.

1. Place the pad onto the pad holder.
2. Wrap the band around the pad holder.
3. Place the pad holder into the soft case included with the device.

**Note:**

- The device, pad and band on the pad holder can be stored together.
- Do not keep the device in areas subject to direct sunlight, high or low temperatures, humid areas, near fire, vibration or shock.

Storage temperature

+32 - +104°F (0 - +40°C), 30 - 80% relative humidity

## Read the Focus (PM710-M/PM710-L) instruction manual for more detail.

If you have any questions or need assistance, please call 1-800-634-4350 or visit [OmronHealthcare.com](http://OmronHealthcare.com).



**OMRON HEALTHCARE Co., Ltd.**

53, Kunotsubo, Terado-cho, Muko, Kyoto, 617-0002 JAPAN

Distributed by: **OMRON HEALTHCARE, INC.**

1925 West Field Court, Lake Forest, IL 60045 U.S.A.

[OmronHealthcare.com](http://OmronHealthcare.com)