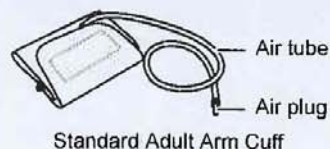
# OMRON®

Thank you for purchasing the OMRON® H-CR24 Arm Cuff. This arm cuff is intended for use with OMRON blood pressure monitors designed to be used with a D-ring Cuff. The H-CR24 D-ring Cuff fits an arm circumference range of 9" to 13" (22cm to 32cm).

 Refer to the instruction manual provided with the blood pressure monitor for complete instructions on operating the device.


## INSTRUCTIONS FOR USE

1. Two air plugs are included with this arm cuff.  
Select the air plug that matches the air plug on the original cuff that was included with the purchase of the OMRON blood pressure monitor.
2. Insert the matching air plug securely into the end of the air tube on the new cuff.
3. Remove the old cuff from the blood pressure monitor and discard.
4. Connect the new cuff to the unit by inserting the air plug into the air jack on the unit.

## HOW TO APPLY THE ARM CUFF

1. Make sure the air plug is securely inserted into the blood pressure monitor.
2. Remove tight-fitting clothing from your upper left arm.
3. Apply the cuff to your upper left arm so the arrow is centered on the inside of your arm and aligned with your middle finger. The air tube runs down the inside of your arm. The bottom of the cuff should be approximately 1/2" above your elbow.
4. Wrap the cuff firmly in place around your arm using the cloth fastening.

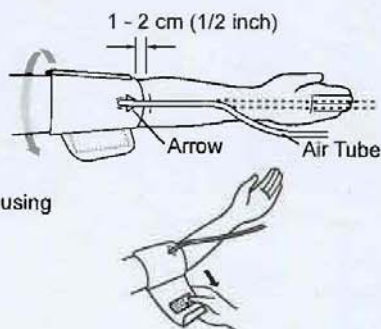Toll-Free Customer Service  
1-800-634-4350

Manufactured for: OMRON HEALTHCARE Co., Ltd.  
53, Kunotsubo, Terado-cho, Muko, Kyoto, 617-0002 JAPAN  
Distributed by: OMRON HEALTHCARE, INC.  
1925 West Field Court, Lake Forest, IL 60045 U.S.A.

www.omronhealthcare.com  
© 2013 OMRON HEALTHCARE, INC.

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Fabriqué pour: OMRON HEALTHCARE Co., Ltd.
Fabricado para: OMRON HEALTHCARE Co., Ltd.