

Thank you for purchasing the OMRON® Contoured Wide Range Arm Cuff (H-RML30). This arm cuff is intended for use with OMRON blood pressure monitor models: BP710, BP742. The H-RML30 Cuff fits an arm circumference range of 9" to 17" (22cm to 42cm).



Refer to the instruction manual provided with the blood pressure monitor for complete instructions on operating the device.


INSTRUCTIONS FOR USE

1. Remove the old cuff from the blood pressure monitor and discard.
2. Connect the new cuff to the unit by inserting the air plug into the air jack on the unit.



HOW TO APPLY THE ARM CUFF

1. Make sure the air plug is securely inserted into the blood pressure monitor.
2. Remove tight-fitting clothing from your upper left arm.
3. Apply the cuff to your upper left arm so the arrow is centered on the inside of your arm and aligned with your middle finger. The air tube runs down the inside of your arm. The bottom of the cuff should be approximately 1/2" (1 - 2 cm) above your elbow.
4. Wrap the cuff firmly in place around your arm using the cloth fastening.

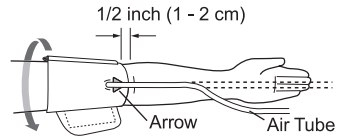

LIMITED WARRANTY

Your H-RML30 cuff is warranted to be free from defects in materials and workmanship within 1 year from the date of purchase, when this cuff is used in accordance with the instructions provided with this cuff. The above warranty extends only to the original retail purchaser. We will, at our option, replace without charge any arm cuff covered by the above warranty. Replacement is our only responsibility and your only remedy under the above warranty. To obtain warranty service contact Customer Service by calling 1-800-634-4350 for the address of the inspection center, the return shipping and handling fee, and further instructions.

THE FOREGOING IS THE SOLE WARRANTY PROVIDED BY OMRON IN CONNECTION WITH THIS PRODUCT, AND OMRON HEREBY DISCLAIMS ANY OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. IMPLIED WARRANTIES AND OTHER TERMS THAT MAY BE IMPOSED BY LAW, IF ANY, ARE LIMITED IN DURATION TO THE PERIOD OF THE ABOVE EXPRESS WARRANTY.

OMRON SHALL NOT BE LIABLE FOR LOSS OF USE OR ANY OTHER SPECIAL, INCIDENTAL, CONSEQUENTIAL OR INDIRECT COSTS, EXPENSES OR DAMAGES.

This warranty provides you with specific legal rights, and you may have other rights that vary by jurisdiction. Because of special local requirements, some of the above limitations and exclusions may not apply to you.

Toll free: 1-800-634-4350
 Manufactured for: OMRON HEALTHCARE Co., Ltd.
 53, Kunotsubo, Terado-cho, Muko, Kyoto, 617-0002 JAPAN
 Distributed by: OMRON HEALTHCARE, INC.
 1925 West Field Court, Lake Forest, IL 60045 U.S.A.

www.omronhealthcare.com
 © 2014 OMRON HEALTHCARE, INC.

Made in China

0187939-6A
 HEM-RML30-Z