# OMRON®

## IMPORTANT SAFETY INSTRUCTIONS SAVE THESE INSTRUCTIONS

### AC Adapter for Omron® Digital Blood Pressure Monitor Model HEM-ADPTW5

Thank you for purchasing the Omron® HEM-ADPTW5 AC Adapter.

 Please read this instruction manual thoroughly before using the unit.

Please keep for future reference.


#### -DANGER-

#### TO REDUCE THE RISK OF FIRE OR ELECTRIC SHOCK, CAREFULLY FOLLOW THESE INSTRUCTIONS.

-  For connection to a power supply not in the U.S.A, use an attachment plug adapter of the proper configuration for the power outlet.
-  The power unit (AC Adapter) is intended to be correctly oriented in a vertical or floor mount position.

#### TO CONNECT THE AC ADAPTER

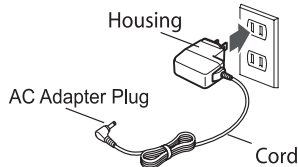- Insert the AC Adapter Plug into the AC Adapter Jack on the monitor.
- Plug the AC Adapter into a 120V AC outlet (60 cycles).

#### TO DISCONNECT THE AC ADAPTER

- Hold and pull the Housing to remove the AC Adapter from the electrical outlet. Do not remove by pulling the cord.
- Remove the AC Adapter plug from the monitor.

#### WARNING

-  Do not plug or unplug the power cord into the electrical outlet with wet hands.
-  Do not overload power outlets. Plug the device into the appropriate voltage outlet.

Manufactured for :  
OMRON HEALTHCARE Co., Ltd.  
53 Kunotsubo, Terado-cho,  
Muko, Kyoto, 617-0002 JAPAN

Distributed by :  
OMRON HEALTHCARE, INC.  
1925 West Field Court,  
Lake Forest, IL 60045 U.S.A.

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