OMRON®

### PMGELS Instructions for Use

Three Pair of PMGELS  
(surface area 40.5 mm x 40.5 mm; shelf life 3 years unopened)  
**For use with Omron's Heat Pain Pro™** only (PM311)  
Omronhealthcare.com  
Single Patient Use Only

#### SAVE THESE INSTRUCTIONS

 Please read this thoroughly before using the gels.

#### INTRODUCTION

##### Intended Use:

The gel refills are only compatible with the Heat Pain Pro's cord with pads. The gels are placed onto the Heat Pain Pro's cord with pads and is intended for use as a re-usable, conductive adhesive gel between the patient's skin and the Electrical Stimulator. Together, the gels with the cord and pads should be applied on normal, healthy, dry, clean skin of adults only.

##### Recommended Use:

3 sessions a day, 30 minutes a session maximum, sessions should not be consecutive.

#### IMPORTANT SAFETY INFORMATION

**CONSULT PM 311 Instruction Manual for additional information.**

##### CONTRAINDICATION :

Do not use this device if you have a cardiac pacemaker, implanted defibrillator, or other implanted metallic or electronic device. Such use could cause electric shock, burns, electrical interference, or death.

- ⚠ WARNING** Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
- ⚠ PRECAUTIONS** Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.

##### WARNING:

- ⚠** Do not place the pads over your neck because this cause severe muscle spasms resulting in closure of your airway, difficulty in breathing, or adverse effects on heart rhythm or blood pressure.
- ⚠** Do not place the pads across your chest because the introduction of electrical current into the chest may cause rhythm disturbances to your heart, which could be lethal.
- ⚠** Do not place the pads over open wounds or rashes, or over swollen, red, infected, or inflamed areas or skin eruptions (e.g., phlebitis, thrombophlebitis, varicose veins).
- ⚠** Do not place the pads over, or in proximity to, cancerous lesions.
- ⚠** Do not place the pads on persons incapable of expressing their thoughts or intentions.
- ⚠** Do not place the pads on persons incapable of operating the unit by themselves.
- ⚠** Do not place the pads while sleeping.
- ⚠** Do not place the pads while driving, operating machinery, or during any activity in which electrical stimulation can put you at risk of injury.
- ⚠** Do not place the pads on infants, toddlers, children, or pregnant women.
- ⚠** Use carefully. May cause serious burns.
- ⚠** Do not use over sensitive skin areas or in the presence of poor circulation.
- ⚠** Do not use this these gel pads during exercise, while sweating, when using heating pad or under a blanket.
- ⚠** If your pain does not improve, becomes more than mild, or continues for more than five days, stop using the device and consult with your physician.

##### PRECAUTIONS:

- ⚠** The long-term effects of electrical stimulation are unknown.
- ⚠** The size, shape and type of pads may affect the safety and effectiveness of electrical stimulation.
- ⚠** The electrical performance characteristics of pads may affect the safety and effectiveness of electrical stimulation.
- ⚠** Using pads that are too small, or incorrectly applied, could result in discomfort or skin burns.
- ⚠** Use caution if stimulation is applied over areas of skin that lack normal sensation.
- ⚠** Since the effects of stimulation of the brain are unknown, stimulation should not be applied across your head and electrodes should not be placed on opposite sides of your head.
- ⚠** Do not use on wet skin (from sweating, bathing, high humidity).
- ⚠** If the product feels too hot, stop use immediately.

##### ADVERSE REACTIONS:

- You may experience skin irritation and burns beneath the stimulation electrodes applied to your skin.
- You may experience headache and other painful sensations during or following the application of electrical stimulation near your eyes and to your head and face.

#### HOW TO USE THE GELS

##### Reminder:

- The cord with pads do not work correctly without the PMGELS. For safety and effectiveness, they must be used together.
  - Before use, inspect the cord with pads for open wires or any damage. If damaged, do not use and replace immediately because it could cause skin burns or fire.
  - Clean and dry affected area so it is free of all lotions/oils/sweat. The PMGEL with cord and pads should be applied only to normal, intact, clean, healthy skin.
1. Remove gels from sealed plastic container.
  2. Peel off the thin plastic film that is blank.
  3. Place this sticky gel side over the black rectangular area of the Heat Pain Pro's cord with pad.
  4. Make sure the gels lay evenly and securely with no bubbles, bending or missing pieces. Check for damage and check the adhesion before each therapy session.
  5. Next, peel off the other thin plastic film that says "SKIN".
  6. Place the remaining sticky gel side around the area of pain at least 1 inch apart (Do not let them touch). Place the gel with the cord and pads onto normal, clean, dry healthy skin that is not experiencing any swelling or inflammation.

##### Note:

- Do not share the same gels with another person because of the risks of adverse skin reactions and to prevent the spread of disease.
- Do not use gel pads after Exp. date.

#### WHEN TO REPLACE THE GELS

If the gel no longer sticks firmly to your skin, or if more than 25% of the gel is not in contact with your skin, replace the gel. (The cord with pads is permanent, do not discard). **DO NOT CLEAN THE GELS. DISCARD AND REPLACE WITH A NEW GEL.**

- Try not to bend or fold the cord with pads with PMGELS during therapy or while storing.
- In between uses, keep the PMGELS and cord with pads together and place the sticky side down onto either side of the plastic pad holder.
- Wrap the cord around the plastic pad holder.
- With proper storage and use, you may extend the life of your PMGELS to 30 uses. The life of the PMGELS may vary due to skin condition and storage condition.
- Store in a cool, dry place.
- Operating temperature / humidity / air pressure: +50 – +104 degrees F (+10 – +40 degrees C) / 30 – 80% RH / 700 to 1060 hPa
- Storage temperature / humidity / air pressure: +32 – +104 degrees F (+0 – +40 degrees C) / 30 – 80% RH / 700 to 1060 hPa
- With proper storage and use, you may extend the life of your pads.

##### KEEP OUT OF REACH OF INFANTS, TODDLERS, OR CHILDREN

Questions? 1-800-634-4350

Manufactured for OMRON HEALTHCARE, Co., Ltd.

PMGELS Packaged in China with goods made in Japan

Distributed by OMRON HEALTHCARE, INC.

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©2016 OMRON HEALTHCARE, INC.
Fabricado para OMRON HEALTHCARE, Co., Ltd.